



FreeStyle
Libre 2



Introducing
FreeStyle Libre 2:
next generation
CGM.

FreeStyle Libre 2 comes with the same
innovative features as FreeStyle Libre:



Zero finger pricks[†]

Always know your glucose level and where it's headed without the need for routine finger pricks.[†]



Powerfully simple

Easy to use¹, painless² to apply and comfortable² to wear for up to 14 days. All the features of CGM without the fuss, wrapped up in a small and discreet sensor.



No calibration

No calibration or input of sensor code required. Simply plug and play.

PLUS additional enhancements:



Alarms[#]

Optional glucose alarms let you know the minute your glucose is too low or too high.



Improved accuracy³

Excellent accuracy overall, even in the low glucose range where it matters most.

*Data based on the number of users worldwide for FreeStyle Libre family of personal CGMs compared to the number of users for other leading personal CGM brands and based on CGM sales dollars compared to other leading personal CGM brands. [†]Finger pricks are required if glucose readings and alarms do not match symptoms or expectations. [#]Notifications will only be received when alarms are turned on and the sensor is within 6 meters of the reading device.

1. Varughese, B. et al. Diabetes Stoffw Herz 2021; 30: 231-242. 2. Haak T. et al. Diabetes Ther 2017; 8(1): 55-73. DOI: 10.1007/s13300-016-0223-6 3. Alva S. et al. J Diabetes Sci Technol 2022;16(1): 70-77. (ePub 2020). DOI: 10.1177/1932296820958754



Abbott
life. to the fullest.

SUPPORT:

Learn how to get the most out of FreeStyle Libre 2 with our short video guides.

Visit [FreeStyleLibre.co.nz](https://www.FreeStyleLibre.co.nz) or scan the QR code to watch:



 **0800 106 100**
FreeStyle Libre Customer Care

[FreeStyleLibre.co.nz](https://www.FreeStyleLibre.co.nz)

How to upgrade from FreeStyle Libre to FreeStyle Libre 2:

Order your FreeStyle Libre 2 sensors from [FreeStyleLibre.co.nz](https://www.FreeStyleLibre.co.nz)



If you are using the FreeStyle LibreLink app:[§]

Continue to use your phone to read the FreeStyle Libre 2 sensor, and enjoy easy monitoring with optional glucose alarms.

FreeStyle LibreLink app notifications may be mirrored on compatible smartwatches.[§]

To learn how to set glucose alarms on the FreeStyle LibreLink app, scan the QR code:



If you are using the FreeStyle Libre reader:

Your FreeStyle Libre reader will **not** read the FreeStyle Libre 2 sensor.

Download the FreeStyle Libre app onto your phone[§] by scanning the QR code or searching 'FreeStyle LibreLink - NZ' on the Apple App Store or Google Play.

To obtain a FreeStyle Libre 2 reader, visit [FreeStyleLibre.co.nz](https://www.FreeStyleLibre.co.nz)



[§]The FreeStyle LibreLink app is only compatible with certain mobile devices and operating systems. Please check the website for more information about device compatibility before using the app. Use of FreeStyle LibreLink requires registration with LibreView.

The FreeStyle Libre Flash Glucose Monitoring System and the FreeStyle Libre 2 Flash Glucose Monitoring system is indicated for measuring interstitial fluid glucose levels in people (aged 4 and older) with insulin-dependent diabetes. The indication for children (age 4 – 17) is limited to those who are supervised by a caregiver who is at least 18 years of age. Always read the instructions for use. The system must not be used with automated insulin dosing (AID) systems, including closed-loop and insulin suspend systems. The sensor must be removed prior to Magnetic Resonance Imaging (MRI).

FreeStyle, Libre, and related brand marks are marks of Abbott. Information contained herein is for distribution outside of the USA only. Mediray New Zealand, 53-55 Paul Matthews Road, Albany, Auckland 0632. www.mediray.co.nz. NZBN 9429041039915. ADC-63774 V1.0