




FreeStyle Libre sensor adhesion guide

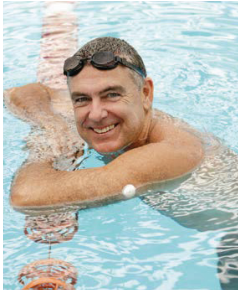

Sensors stick better when you wash, clean and dry your skin before application

1. WASH

Only use **non-moisturising, fragrance-free soaps** to wash the area where you will apply the sensor

2. CLEAN

Use an **alcohol wipe** to clean the site and then allow skin to air dry

3. DRY

Allow your skin to **fully dry** before applying the sensor. This is especially important if you plan to apply the sensor after you shower or swim

BEFORE YOU APPLY THE SENSOR:

Be sure to choose an approved application site - the back of your upper arm

Do not use body lotion or cream where you'll apply the sensor, as they may leave an oily residue on your skin

Check where you are intending to apply the sensor and choose the least hairy site



Why prick, when you can scan?*



This product is indicated for measuring interstitial fluid glucose levels in people (age 4 and older) with insulin-requiring diabetes.

*Scanning the sensor to obtain glucose values does not require lancets. A finger prick test using a blood glucose meter is required during times of rapidly changing glucose levels when interstitial fluid glucose levels may not accurately reflect blood glucose levels or if hypoglycaemia or impending hypoglycaemia is reported by the System or when symptoms do not match the System readings.



MORE INFORMATION

Check out the User Guide that comes with your FreeStyle Libre sensor or visit FreeStyleLibre.com.au

 **0800 106 100**

FreeStyle Libre Customer Care toll-free in New Zealand



Tips to keep your sensor in place



EASY DOES IT

Be careful not to catch your sensor on a doorway, car door, seat belt, or furniture edges



PAT DRY

After a shower or swim, take extra care when towelling off to avoid catching or pulling off your sensor



SLOW DOWN

When dressing or undressing, be careful that you don't catch your clothing on the sensor



HANDS OFF

Try not to play, pull, or touch the sensor while wearing it

Add extra stickiness with these products¹

TORBOT SKIN TAC™

A latex-free “tacky” skin barrier

.....

SKIN-PREP PROTECTIVE WIPES™

Helps tape and films adhere to the skin

.....

MASTISOL LIQUID ADHESIVE®

A clear, non-irritating liquid adhesive that secures dressings

.....

MASTISOL LIQUID ADHESIVE®

A clear, non-irritating liquid adhesive that secures dressings



1. Check the sensor adhesion daily and if it is lifting, apply a medical grade adhesive, bandage or tape according to the manufacturer's instructions. Make sure the opening / hole in the centre of the sensor is not covered.

Abbott Diabetes Care is not affiliated with the manufacturers of the products listed. Reference to third-party products does not constitute or imply an endorsement, recommendation, sponsorship or favouring of any product or manufacturer. Abbott Diabetes Care is not responsible for the completeness or accuracy of any information regarding third-party products. Abbott Diabetes Care makes no representations, expressed or implied, regarding third-party products or their manufacturers, quality or suitability for you. Manufacturer's instructions for use of each product should be followed.

This product is indicated for measuring interstitial fluid glucose levels in people (age 4 and older) with insulin-requiring diabetes. The indication for children (age 4 - 17) is limited to those who are supervised by a caregiver who is at least 18 years of age. Always read the instructions for use. The sensor must be removed prior to Magnetic Resonance Imaging (MRI).